



Title : Modifying Configuration file on the Rigel 288 via Med-eBase

**To send to PC:**

1. On the 288, Press F4
2. Select 'Data Transfer'
3. Select 'Configuration Data'
4. Start Med-eBase software on PC
5. From menu select 'Tools'
6. Select 'configuration Data'
7. Select 'Receive'
8. Select Test Instrument and Com Port, click OK
9. Enter filename and location you wish to use
10. You should now see 'Start Transmission' Box
11. Back on the 288, press 'Send'
12. File will be transferred to PC.
13. From the directory you chose, you can now open and edit the .txt file which was saved. (use notepad or similar)
14. The titles correspond as follows:

Trace 1 = Service Code

Trace 2 = Site

Trace 3 = Location

Trace 4 = Make

Trace 5 = Model

Trace 6 = Description

Trace 7 = Serial Number

Trace 8 = Client

You can now edit the file adding data under the required titles, e.g. Phillips, GE, Welch Allyn etc under Trace 4.

Ward 2, Ward10, IT etc under Trace 3.

DO NOT delete the Trace titles in brackets.

15. When finished, save the file, overwriting the original or as new.



**To Send to the Rigel 288**

1. On the 288, Press F4
2. Select 'Data Transfer'
3. Select 'Configuration Data'
4. Start Med-eBase software on PC
5. From menu select 'Tools'
6. Select 'configuration Data'
7. Select 'Send'
8. Select Test Instrument and Com Port, click OK
9. Choose directory and filename you wish to send
10. Click 'Open'
11. New file will be sent to the 288
12. The screen on the 288 should read 'Valid data Received'
13. END.

End.