



RIGEL MEDICAL
GMC-INSTRUMENTS GROUP

How do I download test sequences to the Uni-Therm?

The file download has the most common test sequences available for the Uni-Therm. See below for the full list.

How to download the test sequences:

1. Download the file to an empty memory stick.
2. Connect the memory stick to the Uni-Therm USB slot.
3. Go to Menu-Data-Transfer Data- select Import test sequences using the F2 button- Select Memory stick in the port area using F3, then select F4 to start the transfer.
4. To make sure you have successfully downloaded the test sequences- from the home page press Menu- Set Up - Test Sequences. You will see the list of test sequences available.

Please note that we are not responsible for the use of these test sequences and they should be used in conjunction with the OEM service procedure.

Bovie Aaron 950

Bovie IDS Series

Conmed CB200 Beamer plus Argon

Conmed CE6000 Beamer

Conmed Hyfrecator 2000

Conmed System 5000
Conmed System 7550
Covidien Valleylab Force 2
Covidien Valleylab Force FX
Covidien Valleylab FT10
ERBE 300 VA
ERBE ICC 200
ERBE ICC 300
ERBE ICC 350
ERBE VIO 200S
ERBE VIO 300
ERBE VIO 300D
Medtronic AEX
Medtronic Covidien ValleyLab Force Triad
Medtronic Ligasure
MEGADYNE MEGAPOWER
Olympus ESG-100
Olympus ESG-400
Olympus PSD-20
Olympus PSD-30
Olympus PSD-60
Olympus UES -30
Olympus UES-40
Valley Force 1
ValleyLab FX
Valleylab FX 8C
ValleyLab FX-C
Vesalius
WOLF

If you require more help, please contact us at
<https://www.seaward.com/storage/enquire/>.