
RIGEL MEDICAL
TESTED, TRUSTED... WORLDWIDE.

UniPulse 400

QUICK START GUIDE

Kurzbedienungsanleitung | Guide D'Utilisation Rapide
Guía De Inicio Rápido | Szybki Start | Guida Rapida
快速入门指南

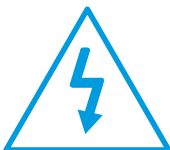Warning of electrical danger!
Warnung vor elektrischer Gefahr!
Avertissement: Danger électrique!
Advertencia de riesgo eléctrico!
Ryzyko porażenia elektrycznego!
Avvertimento di pericolo elettrico!
警告电气危险!

Important, follow the documentation!
Wichtig, Anweisungen befolgen!
Important, suivez la documentation!
Importante, ¡Siga la documentación!
Ważne, postępuj zgodnie z dokumentacją!
Importante, seguire la documentazione!
重要事項，參照文檔！

Important information

Do not use the defibrillator to shock the PACER or ECG terminals.

When connecting to the DEFIB terminals, 4mm shrouded non-stackable plugs must be used.

The DEFIB terminals (red and black) are rated to 5kV/100A/600J MAX.

The PACER terminals (yellow) are rated to 275V MAX.

The UniPulse 400 is designed for use by adequately trained technical personnel only.

No unauthorised user modifications of the instrument.

Do not use the instrument in an application beyond the published specifications.

Use extreme caution when working with voltages greater than 30 Volts, for example a DEFIB shock.

No user serviceable parts.

Do not use the Uni-Pulse 400 with the 18V PSU attached unless it is connected directly to mains power.

Where safe operation of the Uni-Pulse is no longer possible it should be immediately shut down and secured to prevent accidental operation. It must be assumed that safe operation is no longer possible:

- if the instrument or leads show any sign of damage or
- the instrument does not function or
- after long periods of storage under adverse environmental conditions or
- do not use the UniPulse 400 in wet conditions or surrounded in explosive gases or dust.

The UniPulse 400 is designed to analyse Defibrillators, Automated External Defibrillators (AED's) and Pacers, and allows Electrocardiogram (ECG) simulation.

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