



RIGEL MEDICAL
GMC-INSTRUMENTS GROUP

How do I register my Med-eBase license on Windows 7?

- 1) Download Med-eBase from rigelmedical.com. A Med-eBase icon should appear on your desktop.
- 2) Right click on the desktop icon and select "run as administrator", then double clicking on the icon to start the activation process.
- 3) The installation process will then start automatically with Med-eBase licence activation wizard. Follow the onscreen instructions to activate your serial number.
- 4) Activating online requires an internet connection and is the fastest way to activate a product.
- 5) Enter your personal and company information and continue to follow the activation steps to complete the registration process.
- 6) After successful activation close the Med-eBase program then double-click the desktop icon to ensure it opens into the program without requesting the Serial Number to be entered.

Note: An issue is sometimes caused by there being a 'hidden administrator' account on some windows 7 operating systems but when Med-eBase is confirmed to be "run as administrator" the serial number will be permanent when reopening the program.

If you require more help, please contact us at
<https://www.seaward.com/gb/enquire/>.